Vagiwell®

Dilators

- 1 **DE** Medizinische Dehnungsstifte (Dilatatoren) aus Silikon
7 **EN** Medical silicone dilators
13 **FR** Dilateurs médicaux en silicone
19 **ES** Dilatadores vaginales de silicona para uso médico
25 **IT** Dilatatori medici in silicone
31 **EL** Κολληκοί διαστολείς ιατρικού τύπου από σιλικόνη
37 **HU** Orvosi hüvelytágító eszköz szilikonból (dilátor)
43 **DA** Medicinske dilatorer af silikone
49 **NL** Medische pelottes (dilators) van siliconenrubber
55 **NO** Medisinske dilatorer i silikon
61 **FI** Lääkinnälliset silikonista valmistetut laajentimet (dilataattorit)
67 **PL** Medyczne rozszerzaczki silikonowe
73 **CZ** Silikonové lékařské expanzní pomůcky (dilátory)
79 **SV** Medicinska dilatorer av silikon
85 **SK** Medicínske rozťahovacie pomôcky (dilátory) vyrobené zo silikónu
91 **MS** Dilator silikon perubatan

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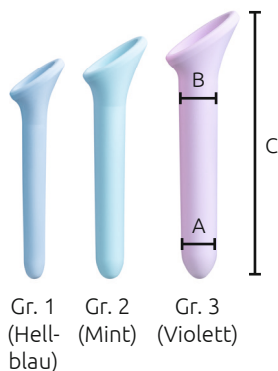
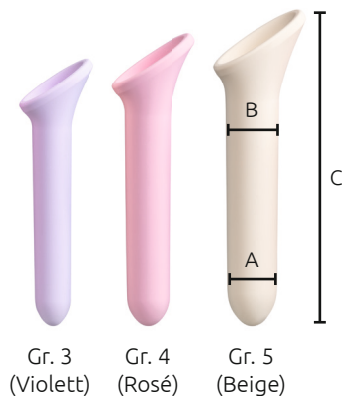
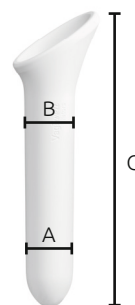
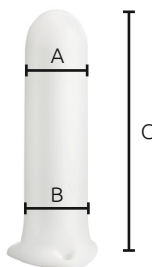
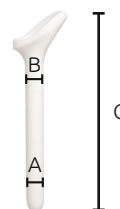
PZN: 19445644

Indications:

- Dyspareunia (recurring or constant genital pain during sexual intercourse) of different causes, e.g.:
 - vaginitis (inflammation of the vagina)
 - adnexitis (combined inflammation of the fallopian tubes, ovaries and surrounding tissue)
 - vaginal stenosis
 - endometriosis
- vaginismus (tightening of the vaginal muscles)
- scarring following surgical cuts to the perineum (episiotomy) or perineal tears
- subsequent to gynaecological operations (e.g. prolapse surgery or cancer operations)
- during and after radiotherapy/chemotherapy
- after vaginal surgery involving tightening of the vagina and scarring
- skin conditions such as certain lichen forms, e.g. lichen sclerosus
- vaginoplasty/neovagina (e.g. during sex reassignment surgery)
- narrowness of the vagina

Stretching exercises with the Vagiwell® dilators can also be useful in making gynaecological examinations or the insertion of tampons less painful.

Application:

The aim of the exercises is to desensitise the vagina by slowly stretching it, thereby making sexual intercourse pain-free (once more) and gynaecological examinations a painless experience.

Do the exercises in a place where you feel comfortable and are relaxed. Make sure that neither the telephone nor anything else will disturb you while you are carrying out the exercises.

1. Prior to use, wash the Vagiwell® dilators with hot water and a mild soap and dry them thoroughly.
2. Find a relaxed sitting or lying position that allows you to spread your legs slightly. For inserting the dilators, we recommend lying flat on your back with your knees bent and your legs spread slightly apart. You can, however, also use the dilators standing up, with one leg on a chair.
3. Lubricate the entrance to the vagina and the dilator with a water-soluble lubricant gel.
4. Breathe calmly. Gently push the dilator into the vagina. You should only insert the dilator as far in as is comfortable for you. Should this be painful, you may be using too much pressure.
5. Leave the dilator in the vagina or in the entrance to the vagina for a few minutes.
6. After that, insert and remove the dilator, without fully taking the dilator out of the vagina. Repeat this movement approx. 5 times. Then slowly start moving the dilator with a slight rotating movement: rotate the dilator 5 times to the right and 5 times to the left. These movements help to gently dilate the vaginal tissue.
7. When you want to finish the stretching exercise, remove the dilator from the vagina. Remain lying down and relax for a few minutes more.

Once you are able to insert one size comfortably, your vagina has stretched far enough for you to be able to use the next largest dilator. When changing from one size to the next one up, the most recently used dilator should first be inserted and left in the vagina for a few minutes. Proceed with the next largest dilator as described above. The stretching exercises may be repeated three to four times a week. Once you are able to comfortably insert the largest dilator, you can consider having sexual intercourse.

Application in cases of lichen sclerosus and vulvodynia:

For women with lichen sclerosus, vulvodynia or lichen planus, a different form of application is recommended.

Before it is actually stretched using the Vagiwell sticks, it is helpful to gently massage the vulva and prepare it for being stretched.

The woman chooses the dilator (size) that suits her personal situation and that she can easily insert into her vagina. She inserts the well-lubricated dilator into the vagina no more than 1 – 2 cm - no deeper - and then begins to stretch the vaginal tissue by gently pressing it outwards from the inside several times with the dilator. Using this stretching movement, she then travels in a circular motion around the entire entrance to the vagina.

It is important to be very careful not to tear or damage the skin. Particular care should be taken at the front of the vaginal entrance to avoid damaging the urethral outlet. It is a good idea to leave this area out at first during the stretching process.

Patience and, if possible, daily stretching are important until the entrance to the vulva becomes more supple and the tissue stretches.

Only when this has been achieved does the user try the same thing with the next largest dilator.

It may take days or weeks before size 5 is also able to be inserted.

Many women also use the even larger "5 LS" dilator later on. Some women have said that they use this size almost once a day to keep the entrance to the vulva supple.

In some countries, there are patient organisations for lichen sclerosus and vulvodynia where you can obtain information and get more tips from users (e.g. www.lichensclerosus.ch). We have taken these tips from the experts at www.lichensclerosus.ch.

We would like to thank them for their help.

Cleaning:

The Vagiwell® dilators are intended for single-patient use. After use, wash the dilator(s) with warm water and mild soap. Make sure that the dilator(s) is/are dry before putting it/them back in the storage pouch (sets only).

Contents:

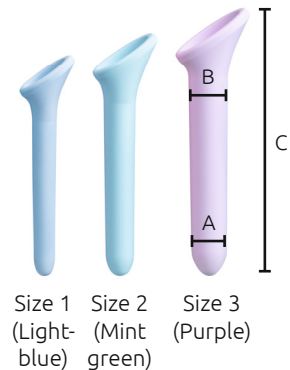
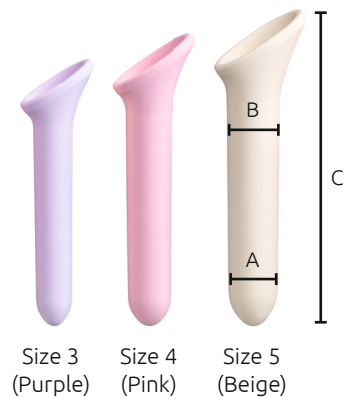
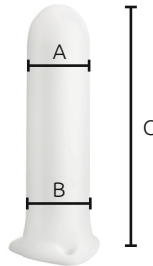
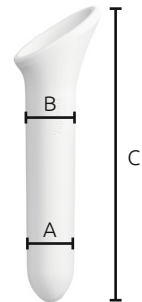
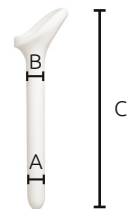
- Vagiwell®
- lubricant, water-soluble
- storage pouch (sets only)
- instructions for use

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KESSEL medintim GmbH